



HemaPrompt™ Easy Instructions

Fecal Occult Blood Test

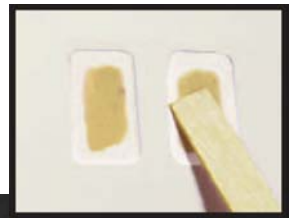


1. **Open the card** so that specimen windows are visible.

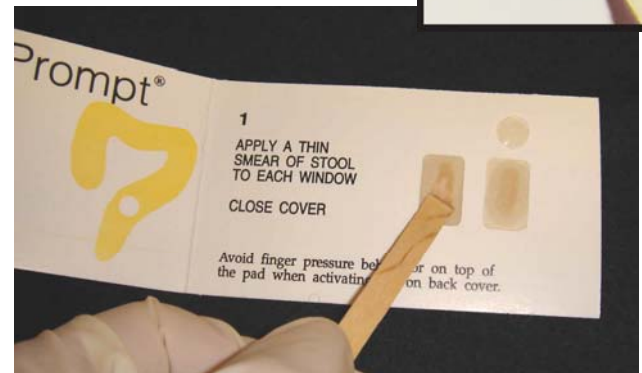


2. **Apply a thin smear of specimen** to each window. ***Do not cover entire window*** (see insert) - leave an outer periphery for contrast. Close card & turn over.




3. **Grip where indicated.** Lift up silver tab so that white developer pad is visible.

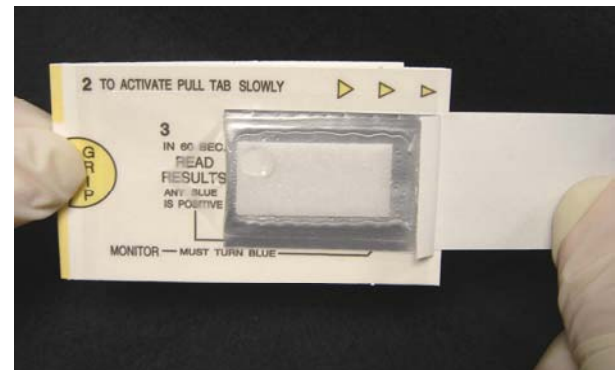

4. **Pull silver tab *SLOWLY*** all the way to the right and completely remove. Developer pad will “flip over” as tab is pulled.

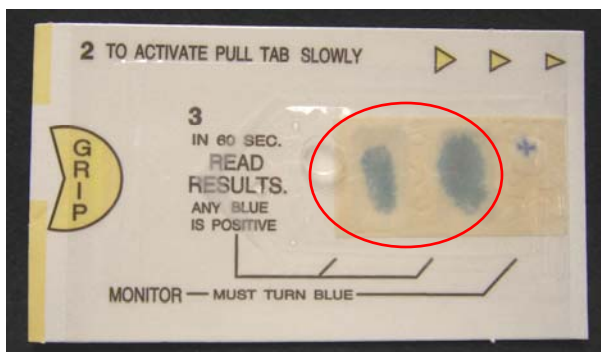

5. **Interpret patient results** after 60 seconds, but before 3 minutes (circled area shown above)
Positive = Any shade of blue*
 (* in either or both windows)
Negative = No blue

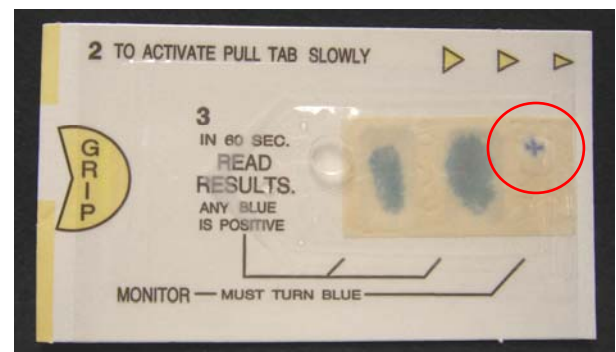

Internal Controls (circled area shown above):
Positive Control: Appearance of Blue Cross
Negative Control: White Background behind Blue Cross

Note: Controls must appear for valid results